ORBIS Cem SE DC

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MD **EU** Medical device

Product description:

ORBIS Cem SE DC is a universal, dual-curing, self-adhesive, luting composite.

It is used for luting indirect restorations made from ceramic, zirconium dioxide, composite and metal as well as for luting posts.

Additional adhesives or conditioners are not required when using **ORBIS Cem SE DC**. **ORBIS Cem SE DC** can be used with any commercial coupling silane (follow manufacturer's instructions).

ORBIS Cem SE DC contains bifunctional methacrylate, acid methacrylate and inorganic fillers. The filler content is 70 % w/w.

ORBIS Cem SE DC is available in the following shade: Universal (U).

The syringe automatically mixes the material and prevents errors and bubbles.

ORBIS Cem SE DC comes with tapered annulae (mixing tips type brown/orange pointed) for easily accessible cavities and restorations as well as intraoral end cannulae (mixing tips type brown/orange blunt Attaching the intraoral tip type B small to the blunt cannula helps access difficult to reach areas and stop drillings.

Indications:

– Definitive luting of inlays, onlays, crowns and bridges (not Maryland bridges) made from ceramic, zirconium dioxide, composites and metal
– Definitive luting of metal, ceramic, zirconium dioxide and fibre-reinforced posts

Contraindications:

ORBIS Cem SE DC contains methacrylates, organic acids, benzoyl peroxide amines and BHT. **ORBIS Cem SE DC** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:

ORBIS Cem SE DC is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

ORBIS Cem SE DC should only be used by a professionally trained dental practitioner.

Use:

Pulp protection:

Areas in close proximity to the pulp should be covered with a hard-setting calcium hydroxide preparation to avoid pulp irritations.

1. Removing the temporary and preparing the cavity

Remove the temporary and thoroughly clean the cavity or prepared core, e.g. using oil- and fluoride-free cleaning paste.

2. Preparation

Check the fit and contact points of the definitive restoration. If silicone has been used for verification, thoroughly remove silicone residues afterwards. Checking the occlusion on glass-ceramic restorations should only be carried out after definitive luting to reduce the risk of fracture. Remove any residue from the surface of the tooth and restoration after completing the try-in. Prepare the restoration according to the manufacturer's instructions and dry it carefully. Dry the surface of the tooth, preferably with either a foam pellet or gentle puffs of air, leaving the dentine slightly moist.

3. Inserting the restoration
Bring material to room temperature before use. Due to technical reasons, there may be slight differences in the filling levels of the syringe chambers prior to first use. Therefore remove the syringe cap and extract material until both chambers deliver uniformly. Then fit a mixing tip type brown/orange pointed or type brown/orange blunt. The notches on the syringe and the mixing tip must line up exactly. Turn the mixing tip 90° clockwise to lock the tip into place. If necessary an intraoral tip type B small can be attached to the end of the mixing tip type brown/orange blunt. To guarantee homogeneous mixing of both pastes, please discard the first 2 mm of the extracted material. After that, apply the material immediately and directly onto the prepared contact surfaces.

Insert the restoration with the aim of leaving a small amount of excess material.

Curing behaviour:

Working time:	90 s (23 °C)
Intraoral working time after insertion:	30 - 60 s
Setting time:	4 min (37 °C)

An evaluation of the curing of **ORBIS Cem SE DC** on mixing pads or similar does not provide valid evidence. This is due to the fact that **ORBIS Cem SE DC**, as is typical for luting composites, does not set hard completely (or not at all, where layers are thin) when exposed to oxygen at room temperature. Completion of curing is guaranteed after insertion of the restoration, at mouth temperature and unaffected by ambient oxygen.

The attached mixing tip is intended for single use only. After use, store the syringe tightly sealed. Check that material flows freely through the openings before reusing.

The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories.

4. Removal of excess material
Remove excess material from difficult to reach areas first. Clean contact points and interproximal areas with dental floss. Excess cement can be removed with a suitable instrument (scaler) either during the gel phase (ca. 60 - 90 s after insertion) or after brief light-curing (1 - 2 s). Larger amounts of excess material are easier to remove. The restoration must be held in position during the polymerisation phase.

As an alternative, any coarse pieces of excess luting composite can be removed with a foam pellet or a similar object immediately after insertion. Leave a small amount of excess material and cover it with a glycerine gel (air-block) or light-cure it.

5. Curing
Polymerisation time can be controlled with additional light-curing. Cure translucent restorations with light for 20 s on each side. Light-curing the luting gap for 10 s on each side is recommended for opaque restorations. The autopolymerisation of **ORBIS Cem SE DC** is completed 4 min after insertion.

6. Finishing

Excess material can be worked immediately after polymerisation. Remove

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