• Gingiva Mask
• Gingiva Mask 3D

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ORBIS Gingiva Mask / Gingiva Mask 3D

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EN Instructions for use

ORBIS Gingiva Mask / Gingiva Mask 3D

Thank you for your confidence in us. By purchasing ORBIS Gingiva Mask/ Gingiva Mask 3D you have made the right decision. Read these in- structions for use carefully prior to use. ORBIS Gingiva Mask/ Gingiva Mask 3D is intended exclusively for dental use. Responsible and careful working is assumed.

Product description:

Both gingiva Mask materials are addition-curing polyvinyl siloxane (VPS) and serve to make laboratory manufactured gingival Masks for precise replication of the gingival profile in the master model. Due to the excellent scannability of the ORBIS Gingiva Mask 3D additional digital data for CAD/CAM systems may be obtained quickly and easily.

Indication:

Gingiva Mask: laboratory manufactured soft gingival Mask for crowns and bridges
Gingiva Mask 3D: laboratory manufactured, hard gingival Mask for dental implant projects
generating digital data for CAD/CAM systems

Dispensing / processing:

Place the cartridge in the 1:1 mixing gun, remove the cap and when using for the first time express material until material is flowing evenly from both orifices. Now place the mixing tip on the orifice (ensure that the colour code of the cap and mixing tip are identical) and ensure that it is on correctly. To close it, turn the tip 90° clockwise. If necessary use Intra-Oral-Tip on the mixing tip. Operate the mixing gun again and dispense material in the desired amount. The used mixing tip stays on the cartridge and acts as a cap. Prior to the next use, remove the used mixing tip and check the orifices for the plug that forms in very rare cases, if necessary, clean and proceed as usual.

Method of application:

Indirect method

- Make a silicone matrix on the uncut die model in the relevant area, such as from ORBIS Tec C-Silicone or ORBIS Tec A-Silicone. After curing, the silicone key is trimmed and one or more injection holes and opposite exit holes are made.
- The area which is to be replicated later with the ORBIS Gingiva Mask/ Gingiva Mask 3D must be removed with a plaster cutter in the desired thickness. Then saw model and grind dies in the gingival area as usual.
- Insulate the silicone matrix with a commercially available suitable insulating agent and allow to dry to achieve a homogeneous surface. Observe the insulating agent manufacturer's specifications.
- Place the silicone key on the model and fix it in place
- Inject ORBIS Gingiva Mask/ Gingiva Mask 3D into the first hole until it comes out the other hole; continue this with all of the holes.
- Alternatively, ORBIS Gingiva Mask/ Gingiva Mask 3D can also be applied directly onto the model and into the matrix, before positioning the silicone matrix. Any previously created drainage channels in the silicone matrix will ensure draining of any excess material.
- After the curing time of approx. 8 min. remove retentions at the holes with a scalpel and detach the key from the ORBIS Gingiva Mask/ Gingiva Mask 3D.
- Remove the press flanges and excess material with a scalpel, scissors or suitable rotating instruments at low speed. The gingival Mask can now be removed from the model and reset again.

Direct method

- Apply a suitable insulating agent evenly to the previously disinfected impression, e.g. with ORBI-Sept IMD Impression Disinfection in the relevant area and allow to dry according to the manufacturer's instructions. Polyether impression material might negatively influence the hardening process of the gingival Mask. The polyether impression should be exposed to air for a minimum of 3 hours time.
- Subsequently inject ORBIS Gingiva Mask/ Gingiva Mask 3D directly into the relevant area of the impression; allow to harden for approx. 8 minutes.
- Pour the impression with suitable dental gypsum and make the model as usual.
- After hardening, cautiously separate the impression from the model.
- Remove pressure markers and any excess material with scalpel or rotary instruments. The gingival Mask may now be removed from the model and reset.
- For ORBIS Gingiva Mask 3D only: To obtain digital data for CAD/CAM systems, simply scan the existing gingival Mask as usual. The addition of scanning powder or spray is not necessary

Working and safety instructions:

- Because mixing is fully automatic, a negative influence on the material because of contact with certain types of gloves, e.g. latex gloves, is impossible. This sensitivity is generally present with addition-curing silicones and must be taken into account when working them.
- Avoid any contact with eyes when using gingiva Mask/ gingiva Mask 3D. In case of contact, immediately rinse with copious amounts of clear water and seek medical care without delay.
- Contact with clothing should be avoided as cured silicones are chemically stable and produce marks that cannot be removed.
- Handle the cartridge material at room temperature (18 °C (64.4 °F) – 25 °C (77 °F) only.
- Storage: store between 5 °C (41 °F) and 27 °C (80.6 °F)
- Shelf life: see package imprint
- Mixing tips and intra-oral tips are intended for single use only.

Technical data and product properties

Technical data (acc. ISO 4823)	ORBIS Gingiva Mask	ORBIS Gingiva Mask 3D
Consistency	Type 3	Type 2
Dosage / mixing time	1:1 / automatic	1:1 / automatic
Working time incl. Mixing time*	90 seconds	90 seconds
Total setting time*	7 – 8 Min.	7 – 8 Min.
Colour	Pink	Pink
Deformation under pressure	< 7,0%	< 3,0%
Recovery from deformation	> 99,90%	> 99,70%
Linear dimensional change	-0,20%	-0,20%
Hardness Shore A	A 35	A 70
Scannable	No	Yes

* at 23 °C (73.4 °F). The time specifications in the upper table relate to a relative humidity of 50 +/- 10 %. In general, higher temperatures accelerate while lower temperatures delay the setting.

Please also note the warning in the safety data sheet.
Information at www.orbis-dental.com
Errors and changes reserved.

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